

CONDOM AMARELLE SAFETY (RED RIBBON)

Colourless and odourless, easy to unroll, moist-coated and with reservoir. Partial proceeds in favour of Deutsche Aidshilfe NRW e. V. and the HOPE Foundation. AMARELLE condoms Safety (Red Ribbon) 36s pcs. and 100 pcs. This condom is available in various lengths and widths, offering the perfect fit for every individual. AMARELLE Safety condoms were developed in cooperation with experts from AIDS service organisations in Germany and offers effective protection against HIV / AIDS when used correctly. As a medical device, AMARELLE Safety condoms protect against unwanted pregnancies.

- MEDICAL DEVICE
- ARE EASY TO REMOVE FROM THE FOIL
- ARE EASY TO UNROLL
- COLOURLESS AND ODOURLESS
- CYLINDRICAL SHAPE
- WITH RESERVOIR
- MOISTENED WITH SILICONE OIL
- LANGUAGES ON 36 AND 100 PIECES BAGS: EN/DE/FR/IT/CZ/PT/ DK/NL/ SV/ES/PL/TR/GR/HU
- LANGUAGES ON 2 PCS. BOX: DE/EN



IMPORTANT INFORMATION BEFORE USING A CONDOM

This product is made of natural rubber latex. People who are allergic to natural rubber should not use this product. Please store in a dry place. Keep away from sunlight. Do not use twice. Medical device class 2b. Please read the instructions for use before use. 1. If extra lubrication is needed, use water-based lubricants. 2. DO NOT USE OIL BASED LUBRICANTS, mineral oils or body lotions, as they may weaken the condom and cause subsequent failure of the product. So can some topical medicines that are applied to the penis or vagina. If you are not sure about the effect of the medicines you're using on condoms, ask your doctor or pharmacist. 3. Do not use the condom, if the rubber material is sticky or brittle. If the individual container is obviously damaged, discard that condom and use a new one from undamaged package. 4. Seek medical assistance as soon as possible, at least within 72 hours, should a condom leak or burst during use. 5. Condoms are for single use only, reused condom could increase the risk of breakage and infection. 6. When used correctly every time you have sex, latex condoms help prevent pregnancy, reduce the risk of transmitting HIV/AIDS and other sexually transmitted infections. 7. No method of contraception can provide 100% protection against pregnancy or transmission of HIV and STIs. If you suspect that you or your partner may have become pregnant or infected, consult your doctor immediately. 8. This product contains natural rubber latex, which may cause allergic reactions including anaphylactic shock. If you are allergic to natural rubber latex products, consult your doctor for advice before use. 9. Store in a cool dry place away from direct sunlight. 10. Keep away from children. 11. There is a risk of choking using condoms orally. 12. The use of additional lubricants during anal sex can reduce the risk of breakage. 13. Condom is a medical device to be used by adult males or females. 14. Report any serious incident occurred from using the device to the manufacturer and the relevant competent authority. 15. These condoms conform to international standards ISO 4074/ EN ISO 4074.

INSTRUCTIONS FOR THE USE OF A CONDOM

(1) Carefully tear off foil pack, making sure not to damage the condom. (2) During removal from foil, avoid damage to the condom by fingernails, jewelry and etc. (3) Squeeze the reservoir end of the condom gently, between the thumb and the forefinger to expel the air out of the tip, while placing the condom over the head of the erect penis. This must be done before any contact occurs between the penis and the partner's body to assist in the prevention of sexually transmitted infections and pregnancy. (4) While still holding the reservoir tip, unroll the condom over the entire length of the erect penis with the other hand. (5) Stop and check, if the condom is slipping off or tightening excessively onto the penis as this might lead to breakage. (6) After ejaculation, hold on the rim of the condom and slowly withdraw the penis, while it is still erect, so that the condom does not slip off. (7) Remove the condom from your penis by sliding it off slowly. It is important not to permit the penis or used condom to touch the vagina area after withdrawal. (8) Wrap the used condom and dispose it hygienically in a closed rubbish bin.

PRODUCT NAME	ART. NO.	UNIT	EAN CODE
AMARELLE KONDOME SAFETY 49mm (RED RIBBON), 36 PCS.	4719989	40	4260231450159
AMARELLE KONDOME SAFETY 49mm (RED RIBBON), 100 PCS.	4719994	20	4260231450241
AMARELLE KONDOME SAFETY 53 mm (RED RIBBON), 36 PCS.	4719990	40	4260231450166
AMARELLE KONDOME SAFETY 54mm (RED RIBBON), 2 PCS. (PAPERBOX)	4719463	50	NO
AMARELLE KONDOME SAFETY 54mm (RED RIBBON), 100 PCS.	4719462	20	4260231451941
AMARELLE KONDOME SAFETY 57mm (RED RIBBON), 36 PCS.	4719996	40	4260231450197
AMARELLE KONDOME SAFETY 57mm (RED RIBBON), 100 PCS.	4719995	20	4260231450203
AMARELLE KONDOME SAFETY 60mm (RED RIBBON), 36 PCS.	4719991	40	4260231451767
AMARELLE KONDOME SAFETY 60mm (RED RIBBON), 100 PCS.	4719987	20	4260231450173
AMARELLE KONDOME SAFETY 64mm (RED RIBBON), 36 PCS.	4719992	40	4260231450227
AMARELLE KONDOME SAFETY 64mm (RED RIBBON), 100 PCS.	4719993	20	4260231450234

• TARIFF NO.: 40141000

ECOaction GmbH, Hermann-Heinrich-Gossen-Straße 4, D-50858 Köln (Cologne), Germany
Phone +49 (0) 2234 9296600, info@ecoaction.de, www.ecoaction.eu